

# WOW

A BIOTECH DENTAL BRAND



## YOUR WOW™ **DISINFECTION PROTOCOL**



BIOTECH DENTAL



# WOW™ DISINFECTION PROTOCOL

In order to enhance the health security of professionals and patients in dental offices, the WOW™ team took the initiative to strengthen their disinfection protocol as well as the WOW™ handpiece.


This new high-level protocol that we will provide with the new handpiece is in total compliance with the disinfecting endoscopes instructions (1). It uses class IIb disinfectants (2) also recommended by the French Union for Oral Health.



IMMERSION



CLEANING

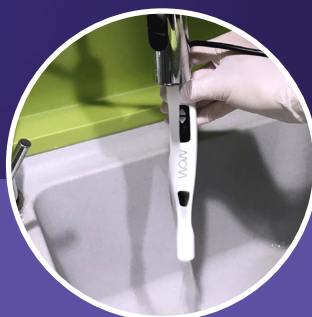

RINSING



DRYING

(1) INSTRUCTION N° DGOS / PF2 / DGS / VSS1 / 2016/220 of July 4, 2016 relating to the treatment of flexible heat-sensitive endoscopes with channels within health and care places published by the Ministry of Social Affairs and Health

(2) according to European Directive 93 / 42 / EC, rule 15 annex IX.



# STEP 1

## Pre-disinfection and cleaning

- ① Between each patient, carry out a pre-disinfection process by completely immersing your handpiece in a detergent / pre-disinfectant solution. Make sure to respect the recommended immersion and dilution time of the chosen detergent / pre-disinfectant product.
- ② Clean by gently rubbing the handpiece to remove any stains associated with the impression-taking.
- ③ Rinse the handpiece with freshwater.
- ④ Gently dry.



# STEP 2

## Total disinfection

- ① Fully immerse your handpiece in a disinfectant solution. Make sure to respect the recommended immersion and dilution time of the chosen disinfectant.
- ② Thoroughly rinse the handpiece with fresh water.
- ③ Gently dry.
- ④ Store your handpiece safe from any contamination.

**The selected products must meet the following standards and regulatory requirements:**

- CE marking class IIb medical device
- Bactericidal standard NF EN 13727, NF EN 14561, BMR according to EN 13727
- Yeasticidal / fungicidal standard: NF EN 13624, NF EN 14562
- Virucidia standard: NF EN 14476: active on HIV-1, PRV (HBV model virus), Herpesvirus, BVDV (HCV model virus), Vaccinia virus.
- Mycobactericidal standard: NF EN 14348, NF EN 14563
- For example, the aniosyme X3 pre-disinfectant / detergent and the 2% steranios disinfectant meet these requirements.



**If you already have a WOW™ intraoral scanner, we can carry out a standard exchange of your handpiece, please contact your sales manager or the after-sales service :  
+33 (0)9 72 17 68 64 - [service@wow-scan.pro](mailto:service@wow-scan.pro)**